

Please read this package insert carefully before use.

Capilia™ MAC Ab ELISA Calibrator

(5 conc. × 0.25 mL)

This product is a calibrator set for Capilia MAC Ab ELISA (an anti-mycobacterium antibody kit).

CONTENTS

Capilia MAC Ab ELISA Calibrator (5 conc. × 0.25 mL) (lyophilized product)

This product comprises 5 different calibrator concentrations.

(The concentration is expressed as U/mL [U: a unit set by our company], labeled on each bottle.)

PREPARATION AND USAGE

After the calibrator is acclimatized to room temperature, add 250 µL of purified water to each of the 5 different concentrations of the calibrators.

Gently stir and mix by inverting. Before use, confirm that each calibrator is completely dissolved.

PRECAUTIONS WHEN USING AND HANDLING

1. The biological materials of human origin contained in this product have been tested negative for HBs antigen, HCV antibodies, HIV 1 antibodies and HIV 2 antibodies.
Because a potential risk of infection or unknown viral contamination cannot be ruled out, this product should be handled with great caution.
2. Do not pipette with the mouth.
3. If this product gets into your eyes or mouth, or comes into contact with your skin, immediately flush with a large quantity of water as an emergency measure. If you feel some abnormality, see a doctor for treatment.
4. Use this product following the operational method described in this package insert. We cannot guarantee results obtained from any other operations and for any other purposes that are not described in this package insert.
5. Use this product immediately after reconstitution. Once opened, store this product in the container tightly closed at 2°C to 8°C and use within 1 week.
6. Do not use any calibrators beyond the expiration date.

STORAGE

Keep in a dark, cool place at 2°C to 8°C.













Keep away from direct sunlight.

INQUIRES



FAX : +81-558-76-0022

GLOSSARY OF SYMBOLS

	CE Marking (European directive 98/79/EC on <i>in vitro</i> diagnostic medical devices)		Authorized representative in the European Community
	<i>In vitro</i> diagnostic medical device		Manufacturer/Manufactured by
	Temperature limitation		Consult instructions for use
	Use by YYYY-MM		Caution, consult accompanying documents.
	Batch code		Keep away from sunlight
	Catalog number		Fragile, handle with care